

CLEANING AND SANITATION

OF ALL LEDERPLAST PRODUCTS



WASHING OR DIRTY REMOVAL

Cleaning with humid and soapy cloth and rinse well with clean water.

SANITATION

A) Spraying the surface with the following solutions:



TESTED SUBSTANCES (UNI EN ISO 105 X12: 2016 modified)

- | | |
|------------------------------------|--------------------------------------|
| > BLEACH 0.5% (WATER 99.5%) | > CHLORHEXIDINE 0.05% (WATER 99.95%) |
| > HYDROGEN PEROXIDE 5% (WATER 95%) | > CHLORAMINE- T 5% (WATER 95%) |
| > ETHYL ALCOHOL 80% (WATER 20%) | > ISOPROPANOL 80% (WATER 20%) |

B) Dry immediately with a soft cloth.



Alcohol component could foster surface wearing and mutate final aspect overtime.
Detergents including additives and alcohol tend to enhance surface wearing and aging process.



PRODUCTS WITH BIOCOTE®

Biocote® component boosts products sanitation giving everlasting antimicrobial effect.

Antibacterial agent gives the product protection 24h between cleans.

**EVERY LEDERPLAST PRODUCT
CAN BE ADDED WITH BIOCOTE®
COMPONENT**

